
Procedure Guide


Scan or visit avitamedical.com for complete instructions for use including important safety information.

RECELL GO Accessories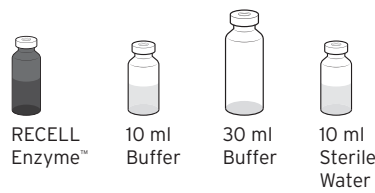


4 x Spray Nozzles

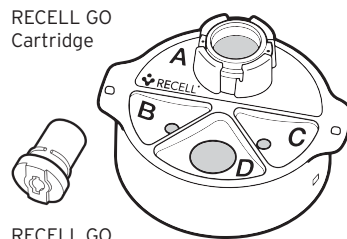RECELL GO Disaggregation Head


RECELL GO Procedure Pack A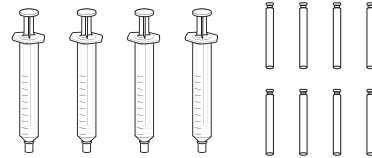

4 x 10 ml Syringes

8 x Blunt Fill Needles

RECELL GO Procedure Pack B

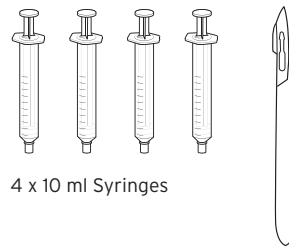4 x 10 ml Syringes

1 x Scalpel

Reusable RECELL GO Processing Device

RECELL GO Processing Device

1 x RECELL GO Processing Device 1 x Power Cord


RECELL GO Preparation Kit

RECELL GO Cartridge Tray	RECELL GO Accessories	RECELL GO Procedure Pack A	RECELL GO Procedure Pack B
1 x RECELL GO Cartridge 1 x Disaggregation Head	1 x RECELL Enzyme 1 x 10 ml Sterile Water 1 x 10 ml Buffer 1 x 30 ml Buffer 4 x Spray Nozzles	8 x Blunt Fill Needles 4 x 10 ml Syringes	4 x 10 ml Syringes 1 x Scalpel

Additional items required: Clean work area, procedure table

The following sterile items: Drape, gauze, saline, blunt forceps, ruler, marker, appropriate wound dressings

1 PREPARE RECELL GO® SYSTEM

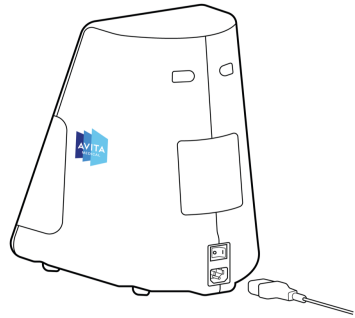The RECELL GO® System is used to prepare Spray-On Skin® Cells from a small sample of the patient's own skin. Ensure that the wound bed is clean, free of necrotic tissue or infection and well vascularized with pinpoint bleeding before proceeding.

Ensure the RECELL GO Processing Device is placed onto a clean and flat area and plugged into a power socket. Switch the device ON at the back. Upon a successful test, the screen will now display:

Total RECELL Cartridges Used XX of 200

The system will then ready itself. Please wait and don't interact with the system at this time.

**Preparing RECELL Processing Device
Please Wait**

The screen will now display:

PLEASE CLEAN

To clean the RECELL GO Processing Device, select the flashing orange CLEAN button.

The screen will now display:

WIPE CLEAN

The shaft will lower and the device is ready for cleaning.

Wipe the equipment with non-woven towelettes containing surface agents that are non-abrasive, non-corrosive and safe for all non-porous surfaces.

Refer to the Instructions for Use for more detailed cleaning instructions.

The screen will display:

Select DONE When Complete

Press the flashing green Done button after the cleaning is completed and the shaft will retract back up.



The screen will now display:

**GO PREPARE
RECELL Cartridge With Enzyme, Buffer and Skin For Processing**

Move to step 2 to start preparation of the sterile RECELL GO Cartridge.

2 PREPARE SINGLE USE CARTRIDGE AND COMPONENTS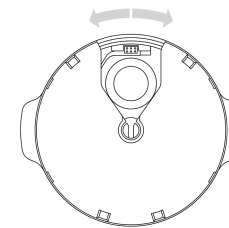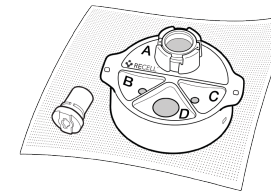

Caution: Do not use sterile RECELL GO® components beyond the stated expiration date indicated on the adhesive Lot # and Expiration Date labels on the outer shelf box.

Open the RECELL GO Preparation Kit and place the Procedure Guide and the component boxes containing the RECELL GO Accessories, including the pouched RECELL Enzyme™, and Procedure Pack items in a non-sterile area. Place the RECELL GO Cartridge Tray in a clean area. The outer shelf box may be discarded.

Remove the lid from the RECELL GO Cartridge Tray and aseptically transfer the RECELL GO Cartridge and Disaggregation Head onto a sterile draped procedure table.



Check the underside of the RECELL GO Cartridge to ensure proper alignment and the pins are centered.

Non-Sterile Preparation Area





Sterile Area

PREPARE ENZYME (WELL A)

- In the non-sterile work area, remove the RECELL Enzyme from the pouch. Remove the cover from the vial marked Enzyme to expose the injection diaphragm. The diaphragm of the Enzyme vial may be wiped with a sterile alcohol wipe and allowed to dry; however, this step is optional.
- Connect a sterile blunt fill needle to a sterile 10 ml syringe and draw up the entire volume of sterile water.
- Inject the entire volume of sterile water into the Enzyme vial. DO NOT USE Buffer at this stage as this may inhibit the Enzyme action.
- Mix gently until dissolved. Do not shake; use care to avoid foaming. Draw the Enzyme back into the syringe.
- Using aseptic technique, dispense the entire volume of Enzyme into Well A. Discard syringe and needle.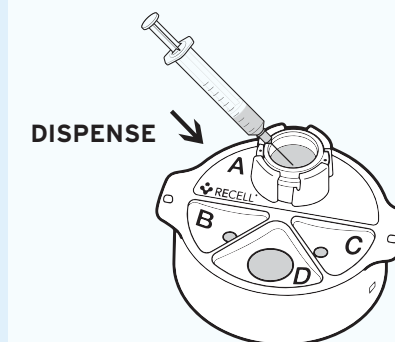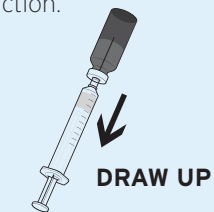

2 PREPARE SINGLE USE CARTRIDGE AND COMPONENTS (continued)

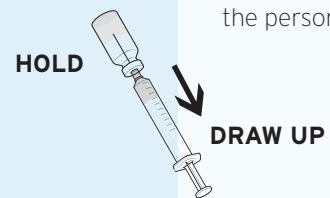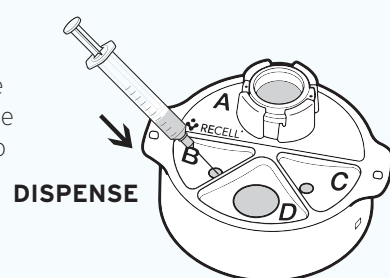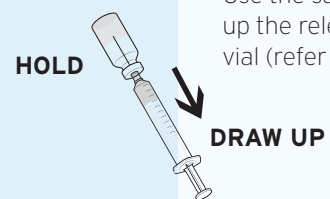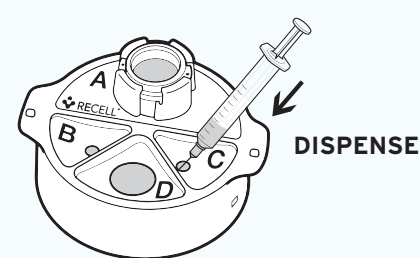Non-Sterile Preparation Area	Sterile Area
PREPARE BUFFER (WELL B)	
<p>The two (2) buffer vials are to remain in the non-sterile preparation area.</p> <p>Using aseptic technique introduce the following items into the sterile field.</p> <ul style="list-style-type: none"> • 1 x 10 ml Syringes • 1 x Blunt Fill Needle 	
<ul style="list-style-type: none"> • Remove the cover from the 10 ml buffer vial. The diaphragm of the buffer vial may be wiped with a sterile alcohol wipe and allow to dry; however, this step is optional. 	<ul style="list-style-type: none"> • Attach the sterile needle to a clean 10 ml syringe.
<ul style="list-style-type: none"> • Hold the buffer vial to allow for the volume to be drawn up by person in the sterile field. 	<ul style="list-style-type: none"> • Draw up the entire volume of buffer (approximately 10 ml) from the vial being held by the person in the non-sterile area.
 <p>HOLD DRAW UP</p>	 <p>DISPENSE</p> <ul style="list-style-type: none"> • Dispense the entire volume of buffer into Well B.
PREPARE BUFFER (WELL C)	
 <p>HOLD DRAW UP</p>	<ul style="list-style-type: none"> • Use the same 10 ml syringe and needle to draw up the relevant volume of buffer from the 30 ml vial (refer to Table 1) and add to Well C.  <p>DISPENSE</p>


2 PREPARE SINGLE USE CARTRIDGE AND COMPONENTS (continued)

Table 1: Guidance on Skin Sample and Buffer Volume for Spray-On Skin Cell Preparation

Treatment Area	Skin Sample Size	Starting Volume of Buffer	Approximate Resultant Spray-on Skin Cells Volume
up to 320 cm ²	2 cm x 2 cm (4 cm ²)	4.5 ml	4 ml
Up to 480 cm ²	3 cm x 2 cm (6 cm ²)	6.5 ml	6 ml
Up to 960 cm ²	2 ea 3 cm x 2 cm (12 cm ²)	12.5 ml	12 ml
Up to 1440 cm ²	3 ea 3 cm x 2 cm (18 cm ²)	18.5 ml	18 ml
Up to 1920 cm ²	4 ea 3 cm x 2 cm (24 cm ²)	24.5 ml	24 ml

Non-Sterile Preparation Area	Sterile Area
PREPARE DELIVERY SET ITEMS (WELL D)	
<p>Introduce all of the remaining Accessories and items from Procedure Pack A and B to the sterile field.</p> <ul style="list-style-type: none"> • 6 x 10 ml Syringes • 4 x Spray Nozzles 	<ul style="list-style-type: none"> • 6 x Blunt Fill Needles • 1 x Scalpel
	<p>RECELL GO Cartridge Set-Up Complete</p>

3 HARVEST SKIN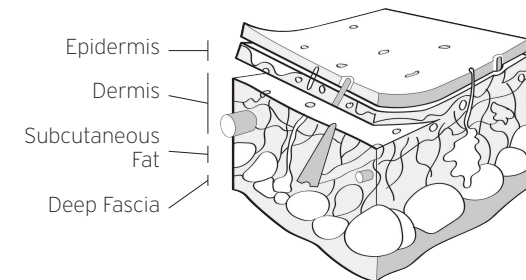

The donor site should be clean, of appropriate depth, and show no evidence of surrounding inflammation or infection. If desired, infiltrate the subcutaneous tissue with a tumescent solution of choice.


The donor site area may be lubricated (e.g. sterile mineral oil) to ease travel of the dermatome.

Harvest thin skin graft at 0.006-0.008 inch (or 0.15-0.20mm). Refer to TABLE 1 for the appropriate sizes.


Use the scalpel provided to cut skin samples into pieces no larger than 3 cm x 2 cm prior to placing in Well A.

A maximum of 24 cm² can be placed in Well A.


4 ADD DONOR SKIN TO RECELL GO CARTRIDGE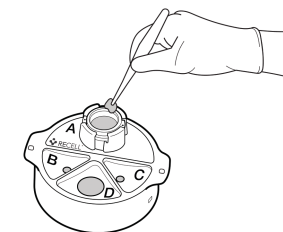

Place donor skin sample(s) into Well A.

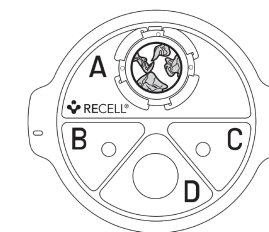
Place donor skin sample(s) Separately from each other in Well A. It does not matter which side of the skin is placed up in Well A.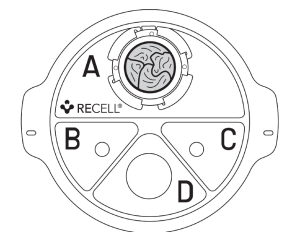


Use blunt forceps to spread out the skin along the base of Well A. **Do not use pointed forceps so as not to pierce the filter in Well A.**

4 ADD DONOR SKIN TO RECELL GO CARTRIDGE (continued)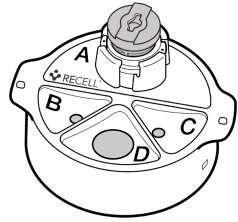

Gently place the Disaggregation Head on top of the skin sample in Well A.

Once the RECELL GO Cartridge is prepared, press the flashing green CONTINUE button.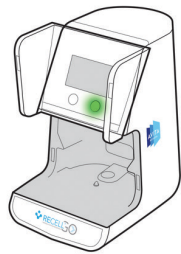


Next the RECELL GO Processing Device screen will display:

VERIFY
RECELL Cartridge Includes Enzyme, Buffer, Skin and Disaggregation Head

Once you have verified the contents of the RECELL GO Cartridge, press the flashing green CONTINUE button.

5 PREPARE THE CELL SUSPENSION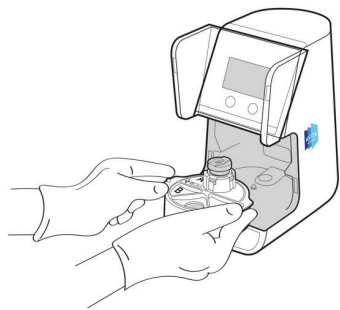

The screen will now display:

INSERT
RECELL Cartridge and Press START

A non-sterile assistant should open the door of the RECELL GO Processing Device.

While maintaining aseptic technique, keep the RECELL GO Cartridge level and with Well A containing the Disaggregation Head at the back of the RECELL GO Cartridge, slide it into the RECELL GO Processing Device until the side tabs on the Cartridge meet the Device.

A non-sterile assistant should close the door and press the flashing green START button to start the skin processing sequence. The screen will display:

RECELL Spray-On Skin Cells Processing
XX Minutes Remaining


Note: If the RECELL GO Cartridge is not properly detected, refer to the troubleshooting section in the Instructions For Use.

The buttons will cease flashing and the screen will count down until the RECELL Spray-On Skin Cells will be ready for collection.

This will take around 35 minutes. Do not open the door while the RECELL Spray-On Skin Cells are processing.

6 PREPARE THE DRESSINGS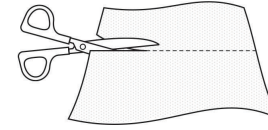

Cut the primary dressing, as required to cover the size of the aftercare wound.

Affix to lower aspect of the wound to help reduce run off.


7 DRAW UP THE PREPARED CELL SUSPENSION

An audible alert will sound and the screen will display:

Process Complete
Aseptically Draw Up
RECELL Spray-On Skin
Cells From Well D

The RECELL Spray-On Skin Cells are ready to be drawn aseptically from Well D.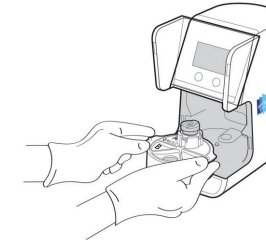

Locate the blunt fill needles and syringes in the sterile draped procedure field. Attach a blunt tip needle to the 10 ml syringe. Return to the RECELL GO Processing Device noting the device is not sterile. A non-sterile assistant should open the door of the device and move the RECELL GO Cartridge forward on the base of the device so that Well D is accessible using aseptic technique. The RECELL GO Cartridge can also be placed in a clean area next to the RECELL GO Processing Device, if desired.

Using aseptic technique, seat the needle in the dimple at the base of Well D. **Mix the cell suspension by drawing up and returning back to Well D.** Draw up the required volume to apply to the treatment area using the additional syringes and needles provided as required.

8 APPLY RECELL SPRAY-ON SKIN™ CELLS

Spray Application: Must have at least 2 ml of cell suspension in the syringe. Connect nozzle to syringe, and check that spray nozzle faces the wound. Hold approximately 10 cm from the most elevated point of the wound and spray.

Drip Application: When applying less than 2 ml of cell suspension, do not remove the syringe from the needle. Starting from the most elevated aspect of the wound, drip cells onto the wound.

9 APPLY DRESSINGS

Apply primary dressings and secondary dressings. Secure dressings with outer bandages.

For Aftercare instructions refer to the RECELL System: Aftercare Instructions for the Healthcare Professional.

10 CLEAN THE RECELL GO PROCESSING DEVICE

Once the procedure is complete, remove the RECELL GO Cartridge and dispose of per facility protocol. It is highly recommended to clean the RECELL GO Processing Device after retrieving from storage and in between uses as part of regular maintenance.

After pressing the flashing green CONTINUE button, the screen will display:

PLEASE CLEAN

To clean the RECELL GO Processing Device, select the flashing orange CLEAN button.

The screen will now display:

WIPE CLEAN

The shaft will lower and the device is ready for cleaning.

Wipe the equipment with non-woven towelettes containing surface agents that are non-abrasive, non-corrosive and safe for all non-porous surfaces.

Refer to the Instructions for Use for more detailed cleaning instructions.

The screen will display:

Select DONE When Complete

Press the green Done button after the cleaning is completed and the shaft will retract back up and the screen will return to display status of RECELL GO Cartridges used. The system is now ready to be stored or used with the next patient.

Caution: Follow standard precautions including use of appropriate Personal Protective Equipment (PPE) while cleaning and reusing the processing device.

11 REUSABLE RECELL GO PROCESSING DEVICE

The RECELL GO Processing Device may be used for (up to) 200 uses with separate single-use RECELL GO Cartridges. The device screen will indicate the number of RECELL GO Cartridges used. Once the count reaches 180 of 200, the RECELL GO Processing Device is within 10% of reaching its limit. It is recommended that you contact your AVITA Medical representative at that point to discuss the appropriate time to replace your RECELL GO Processing Device to ensure uninterrupted patient care. It is important to remember that your RECELL GO Processing Device will not prepare any RECELL GO Cartridges beyond the 200 use limit. Once 200 uses have been reached, the RECELL GO Processing Device MUST be replaced to maintain effective cell harvesting.

AVITA Medical Americas, LLC
28159 Avenue Stanford, Suite 220
Valencia, CA 91355, USA

©2025 AVITA Medical. AVITA Medical®, the AVITA Medical logo, RECELL®, RECELL Enzyme™, RECELL GO® and Spray-On Skin® Cells are trademarks of AVITA Medical. AW-LBLSPEC307REV6

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